


## Instructions

The safe way to draw a blood sample!  
The needle retracts permanently after use.

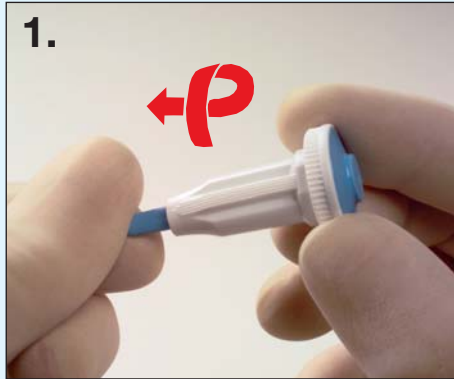

To remove the protective cap, turn and twist in one movement (a full turn).



Place the edge of the lancet approximately 2 mm from the nail.

1. Remove the protective cap.
2. The lancet should rest lightly but firmly against the patients finger.  
Guide marks along the lancet denote the puncture point.
3. **Press the button.**



## HaeMedic

Företagaregatan 18  
S-266 32 Munka Ljungby, Sweden  
Phone: 0431-43 20 30 Fax: 0431-43 00 30  
E-mail: [info@haemedic.se](mailto:info@haemedic.se) [www.haemedic.com](http://www.haemedic.com)  
A company in the Arta Plast group